User's Manual

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Subject : Compressible Limb Therapy System

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
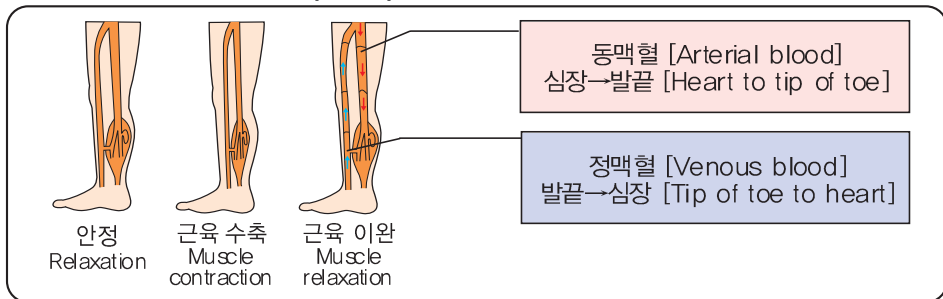
3. 적응증 [Indications use and contradictions] (한국내 적용제외)

Power-Q1000 PREMIUM is intended for use by medical professionals and patients at home, who are under medical supervision, in treating many conditions, such as: Primary lymphedema, Edema following trauma and sport injuries, Postimmobilization edema, Venous insufficiencies, Lymphedema.

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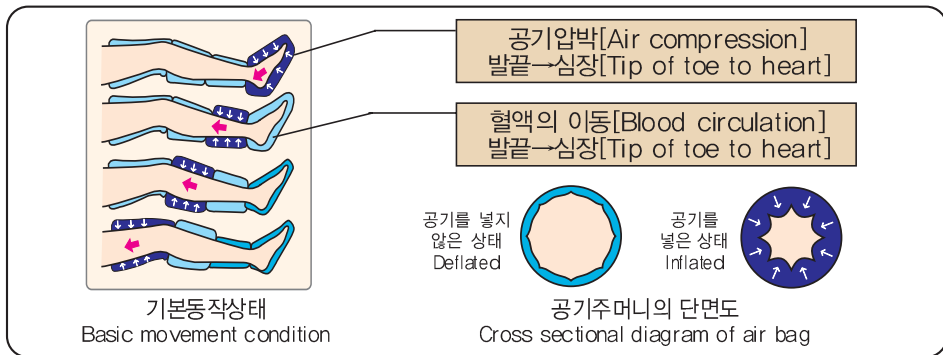금기사항 [Contradictions]

- Do not use this product if you are experiencing inflammation, an infection, pain of unknown origin, or bleeding (internal or external) at or near the site of application or if you have a wound at or near the site of application
- Do not use this product if you have any of the following conditions:
 - Acute pulmonary edema
 - Acute thrombophlebitis
 - Acute congenitive cardia failure
 - Acute infections
 - Deep Vein Thrombosis (DVT)
 - Episodes of Pulmonary embolism
 - Wounds lesions or tumors at or in the vicinity of application
 - Where increased venous and lymphatic return is undesirable
 - Bone fractures or dislocations at or in the vicinity of application

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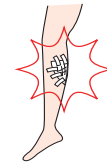
Please consult a physician before using this device if:

- You are pregnant or feel weak
- You have an implant at the site of application
- You have a cardiac pacemaker

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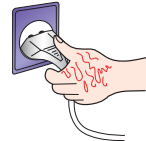
Do not touch the power cord with wet hands.
It may cause electric shock, short circuit or fire.

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
Please check the rating voltage before use.

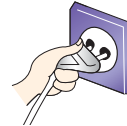It may cause fire, electric shock and/or injuries.

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
Do not put heavy objects on the power cord or press down with table or chair.

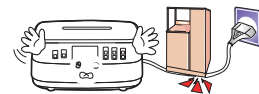It may cause electric shock, short circuit and/or fire.

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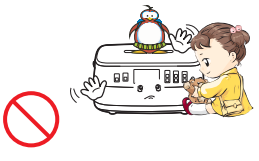
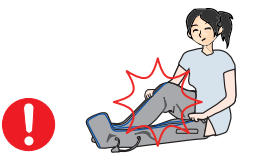
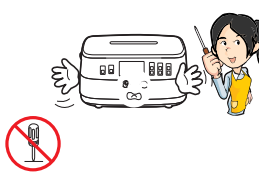
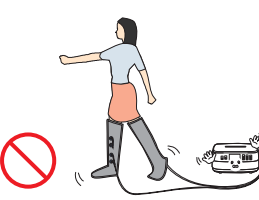
Please careful not to get water or other particles into the device.

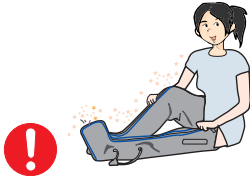
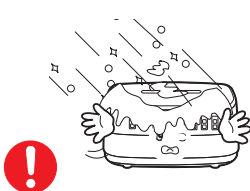
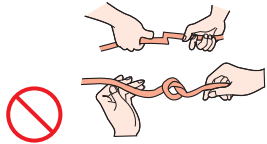
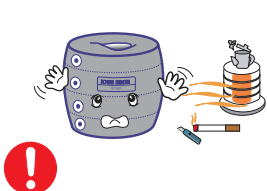
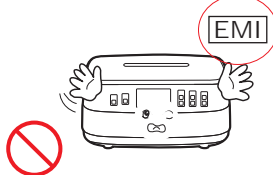
- It may cause electric shock, short circuit and/or fire

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Compressible Limb Therapy System is a pneumatic pressure treatment system That repeats expansion sleeves to help blood circulation and prevent blood clots or clogs.

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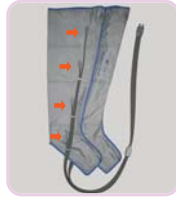
확장지퍼 (좌/우)
Extension zipper
[R/L]

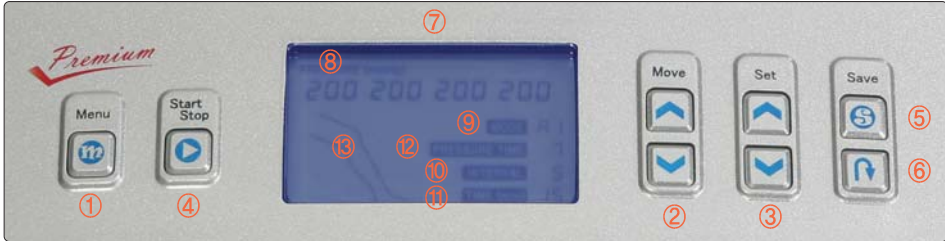
확장지퍼 연결
Extension zipper
connection

확장지퍼 연결 모습
After extension
zipper connection

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Menu selection (if changed by the user)

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If you want to use the basic setting, move to #④.

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Press MENU to change to customer setting.

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Move to menu to be changed using the MOVE button. - Refer to the 11page

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Change the value using SET button. If you want to choose something else, repeat the steps c & d.

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If you have set the setting, press MENU button to get out of the setting change.

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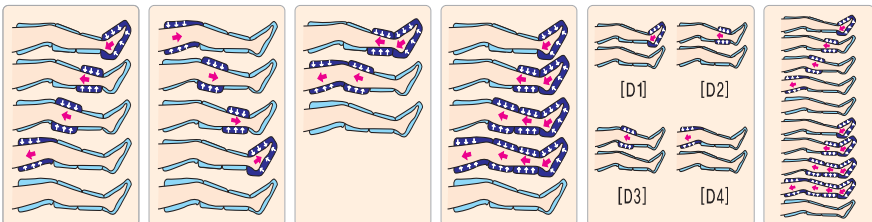
If you want to use the current customer setting again later, press down on the S button (for save) for 10 seconds to save. (If you press button on reset(↵) for 10 seconds, it goes back to the basic setting). If you want to use the setting but don't want to save it, move down to g.

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-Order of pressure is applied to the cuff depending on the mode operating-

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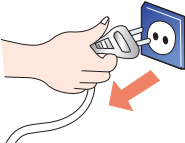
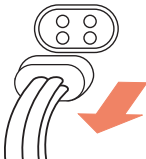
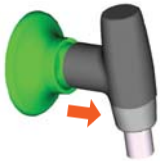
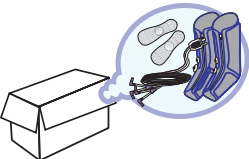
Operate the device by pressing down on start/stop.

-The device stops working automatically if the set time is ended.

사용종료시/When you're done using the product

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작동 보관시의 환경조건/Environments for its operation and custody

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Situations	What to check fors
If the device is not working	<ul style="list-style-type: none"> * Check if the device is plugged in. * Press ON (ON/OFF switch). * Check if the power cord is malfunctioning (Check for the voltage). * Check if timer switch is set.
If it sounds like air is coming out	<ul style="list-style-type: none"> * Check if hose or hose connector is damaged. * Check is hose connector is completely plugged in. * Check if hose is bent or pressed down.
If cuff and waist pad do not get inflated or have no pressure	<ul style="list-style-type: none"> * Check if hose connector is connected appropriately * Check if hose is bent or pressed down by heavy objects * Check is hose is bent or pressed down. * Check if hose connector is plugged into the air injection nozzle until clicking sound.
If air keeps going into the cuff and it sounds like air keeps coming out of it	<ul style="list-style-type: none"> * Check where the air is coming from (which part of cuff) <p>If you cannot spot where the air is coming from, turn off the device and try to operate again.</p> <p>Boot cuff and each pad are accessories.</p>

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A/S : 032-816-0552

WARRANTY

Name of Products	Compressible Limb Therapy System	App licable Mode l	POWER-Q1000 PREMIUM
Date of Purchase		Serial No.	
Agent of Purchase		Warranty Period	Body:1 Year Cuffs(Consumption Goods); 3Months(Work Place)

About Our Service

This is a product that we, Wonjin Mulsan Co., Ltd. manufactured through its own strict quality management and inspection process after obtaining the permission and approval of the Korean Food and Drug Administration and the Safety Authentication System for Electrical Goods. Wonjin Mulsan Co., Ltd. Will provide parts list or maintenance instructions manual of products or more information to perform maintenance or repair the system.

EMC(Electro- Magnetic- Compatibility)

These products are satisfied with EMC standard requirements. Please follow below instructions for the special caution of EMC.

- It will affect the performance of devices for the decline of durability of equipment or the emission of electromagnetic if clients do not follow manufacturer's instructions.
- It will affect normal operation in medical devices by using mobile or communication equipments. The clients need to check normal operation in medical devices before using these equipments.
- Please contact to engineers at Wonjin Mulsan Co., Ltd. In South Korea if you want to know more about EMC.

Paid Services

Please be careful about the fact that the service fee will be charged even within the warranty period if the consumer requests the service due to his/her carelessness or if no defects are found in it.

In case of faults due to the consumer's mistake (When the faults occur due to consumer's carelessness or wrong repair)

- *When the fault occurs due to the wrong use for the electrical capacity.
- *When the fault occurs because the consumer has fallen down the device during the transfer after installation.
- *When the fault occurs because the consumer has used the consumption and optional goods that we did not designated
- *When the fault occurs because other companies' repairman has repaired the product concerned
- *When the fault occurs because the consumer has used the product in the prohibited area(outdoors) or beyond the original use, because the consumer has disassembled, alternated and repaired at his/her discretion.
- *When the consumer has not abide by the matters that require attention in this instruction manual (Please read the matters that require attention carefully.)

Other cases

- *In case of natural disasters(fire, damage from sea wind, flood damage, thunder, lightening, earthquake, etc.)
- *When the consumption goods are being used up or the warranty period has been expired.
- *You need to use authorized parts or supplies which are provided by manufacturer Otherwise, it might be the reason of trouble or accident.

Manufacturer: Wonjin Mulsan, Co., Ltd.
2F, 89, Geomdan-ro, Seo-gu, Incheon, Rep. of KOREA
A/S 82-32-816-0552

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Compressible Limb Therapy System is a pneumatic
pressure treatment system That repeats expansion
sleeves to help blood circulation and prevent blood
clots or clogs.

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Manufacturer : **Wonjin Mulsan, Co., Ltd.**

<http://www.wonjininc.net>

2F, 89, Geomdan-ro, Seo-gu, Incheon, Rep. of KOREA

A / S: 82-32-816-0552