Zweikanal-EMG-Gerät

SineBravo

CE 0123

Inhalt

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Content

Please read the Operators Manual carefully
before using the device and pay attention
to all references !

In the following you'll find all information directly necessary for performing
the treatment with grey background.

Page

I. Biofeedback-device SineBravo

Delivery & Accessories	28
Waste Disposal	28
Technical Data	29
Electromagnetic Compatibility	29
Introduction – What is SineBravo ?	30
Explanation of Symbols	30
Indication	31
Contraindication	31
Warnings	32
Advice on Application of Electrodes	33
Maintenance	34
Warranty	34

II. Display and Operating Elements

Operating Elements	35
Battery Compartment	35
Connections	36
The Display	37
The Info-Line	38

III. Use of SineBravo

Basic Settings	39
Setup Real Time Clock	40
Setup Language	40
Preparation of Therapy	41
Therapy with SineBravo	42

Content

	Page
IV. Additional Options for Configuration of SineBravo	
Configuration of Free Programs	43
Parameters of Free Programs	44
Lock-Button	45
Threshold	46
Setting of threshold before start of treatment	46
Automatical adjustment of threshold during treatment	46
V. Therapy-Control	
Therapy-Control	47
Statistics	48-49
VI. Programs	49
Literature	50
Anatomy	51
Copyright	52

Delivery & Accessories

Included in the delivery:

REF: 010E-087: 1 unit **SineBravo**, 2 cables to connect the electrodes, 1 reference cable, 8 self-adhesive electrodes (40 x 40 mm), 1 stand, 4 batteries type AA, 1 operators manual and 1 storage box

Accessories:

REF

011E-006: 1 unit self adhesive electrodes 40x40 (4 pieces)

011E-007: 1 unit self adhesive electrodes 40x80 (4 pieces)

011E-008: 1 vaginal probe “IncoProbe V”

011E-009: 1 rectal probe “IncoProbe R”

011F-010: 1 vaginal probe “Periform”

011F-011: 1 rectal probe “Anuform”

011F-012: 2 cables to connect the electrodes

011F-037: 1 tube electrode jelly for probes

011F-050: 1 storage box

011F-083: 1 reference cable

011E-095: 1 software for therapists

011F-096: 4 batteries type AA

011E-097: 1 operators manual **SineBravo**

011E-099: 1 fibre optic cable

Waste Disposal

In accordance with European regulations [1], used electrical and electronic equipment may not be given to the unsorted waste anymore. The symbol of the litter bin on wheels indicates the necessity of separate collection.

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Technical Data

1. Dual Channel
2. EMG Range: 0.2 to 2000 μ V RMS (continuous)
3. Sensitivity: 0.1 μ V RMS
4. Selectable Bandpass filter - 3db Bandwidth,
 - a. Wide: 18 Hz to 370 Hz - Reading below 235 microvolts
10 Hz to 370 Hz - Reading above 235 microvolts
 - b. Narrow: 100 Hz to 370 Hz
5. Notch filter: 50 Hz - 33 db
6. Common Mode Rejection Ratio: 130 db Minimum @ 50 Hz
7. Batteries: 4 x AA
8. Work / Rest periods: 2-99 seconds
9. Number of Trials: 2-99
10. Measures of the device: L 141,6 mm / W 74 mm / H 40,4 mm
11. Weight of the device: 270 g (incl. batteries)

Environmental conditions for storage & transportation:

0-93% atmospheric humidity, -25 to +70 degree centigrade.

Environmental conditions for use: 0-93% atmospheric humidity,
+5 to +40 degree centigrade, air pressure 700-1060 hPa.

The **SineBravo** is protected against debris larger than 12.5 mm in diameter from entering the cabinet and is not water resistant and rated IP20.

This device is quality controlled and fulfils the rules and regulations set by the German law for medical products (MPG) as well as the guidelines of the European Community 93/42/EEG for medical products. Therefore this device carries the CE-mark „CE 0123“.

You can find the serial number of this device in the battery compartment.

Manufacturer: MTR+ Vertriebs GmbH, Kamenzer Damm 78,
12249 Berlin

Electromagnetic Compatibility

The device conforms to the electromagnetic compatibility requirements of European Standard EN 60601-1-2 “Medical Electrical Equipment, Part 1: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests”

Introduction – What is SineBravo ?

SineBravo is an easy and inexpensive electromyography (EMG)-biofeedback device. The term biofeedback refers to a method to perceive biological processes with the help of a technical device. This instrument measures physical activity - even if they are not visible or not voluntary - and converts them to optical and/or sound signals. Nevertheless **SineBravo** is not a diagnostic tool, but a two-channel device that can be flexibly used for electromyography.

Since both doctors and therapists are increasingly recognizing the importance of the measurement of muscle activity and the recording of individual progress of patients, EMG is becoming more and more important.

SineBravo is particularly suitable for treatment in the patients' homes.

During treatment it's not necessary to hold **SineBravo**, since it can stand upright on a flat surface. This allows the patient the training or the treatment while observing the success of the individual efforts. The muscle activity is indicated by a yellow-green bargraph on each side of the screen and by numerical values in the bottom-line of the display. At the end of each session, the screen automatically displays the average values of muscle activity in this session. The therapy control memory in the device allows a quick overview of the treatment data at any time, but it's also possible to see the statistics of the last 365 therapy-days. The **SineBravo** can be used with or without connection to a personal computer. The stored treatment data can be transmitted easily later to the PC using a special software available.

Explanation of Symbols

Attention
Warning

Type BF applied parts

Serial-Number
of the product

Keep dry

Manufacturer

2012 - 08

Date of
manufacture

CE-mark: conforms to
essential requirements of
the MDD 93/42/EEC

Batch code

Indications

SineBravo can be used without any help according to the instructions of the treating physician as a treatment for faecal- and bladder-incontinence and to support muscle training for many neurological and orthopedic indications such as incomplete paralysis or in stroke rehabilitation.

Contraindication

During the application of the **SineBravo** it is necessary to take precautions.

SineBravo may not be used in the following cases:

- while operating machinery with dangerous potential
- if patients cannot use the device properly, due to mental or physical limitations

Please don't use a probe:

- during the menstrual period.
- during pregnancy.
- if an inflammation of the vagina, anus or urinary tract is present.
- in the presence of symptoms of a bladder infection.

Warnings

- **SineBravo** is a type BF equipment.
- Do not immerse **SineBravo** into water or any other liquids.
- Do not use **SineBravo** in presence of any flammable gas or air-gas mixture.
- If using rechargeable batteries be sure to use a CE approved battery charger.
Never connect the DuoBravo directly to a battery charger or to any other net-powered equipment.
- To avoid the effects of electromagnetic interference never use the **SineBravo** with less than 4 metres distance to a cellular telephone or near any other powerful radio interference producing equipment that causes electrical signals etc. During EMG-mode the **SineBravo** may be susceptible to strong interfering radio type emissions that may lead to temporally increased EMG microvolt readings.
- The power supply of some laptop computers may cause significant interference affecting the **SineBravo**. As a precautionary measure please make sure that the power cord of the laptop is as far away as possible from the lead wires of the patient.
- If in the case of treatment with the incontinence probes, pain or irritation appear in the area of the pelvis, or bleeding in-between menstruation, interrupt the therapy and consult your doctor.
- Should skin irritation occur whilst using the **SineBravo**, please interrupt the application and inform your doctor, so that an allergic reaction can be excluded.

Please, keep this device stored out of the reach of children.

Advice on Application of Electrodes

- Electrodes which are damaged due to careless handling may not be used again.
- Electrodes with damaged cable insulation (e.g. visible copper wire) may not be used.
- Electrodes with damaged, missing or stained aluminium coating or carbon layer, also with insufficient adhesive gel may not be used.
- Electrodes may not be altered or tampered with (e.g. by scissors or other tools).
- Do not pull the electrodes by force on the electrode cable. To remove the electrodes from the skin, please lift the electrode at the rim towards the middle.
- Do not use electrodes with ointments or creams (especially which enhance the blood flow or act as local anaesthesia).
- One uses special electrodes (probes) and a special electrode gel for the therapy of incontinence.
- Apply the electrodes on clean and unwounded skin only.
- In the case of strong bodily hair growth, the hair should be shaved or shortened with scissors.
- Neither stretch the skin nor the electrodes when applying the electrodes.
- All customary electrodes which comply with the requirements set out in the Council Directive 93/42/EEC relating to the Medical Device Directive can be used, if the connections match the cables of **SineBravo**. We recommend the use of our self-adhesive electrodes with a minimum area of 40x40 mm.
- For treatment of the muscles of the pelvic floor we recommend the use of our vaginal- (IncoProbe V) or rectal-probes (IncoProbe R).
- If you use a probe, please clean it carefully with a mild detergent without alcohol and dry it completely after each use.
- After all electrodes have been removed, wash the skin with mild soap and rinse thoroughly. Then dry your skin.
- All electrodes or probes are made for use of one single person only.

Maintenance

SineBravo has been designed to be maintenance free. Nevertheless, please adhere to the following advice:

If needed, please clean **SineBravo** with a mild detergent, water and a damp cloth. Please don't use alcoholic based detergent.

Do not expose **SineBravo** to extreme humidity and do not submerge under water.

You may clean the cables with a damp cloth. You may also from time to time treat the cables with talcum powder in order to prevent cracks and to further ongoing usage.

Please subject the device to technical safety control in regular intervals in accordance with the local regulations.

After termination of the therapy, **SineBravo** can be re-used by other patients. Before using it please clean with a mild disinfectant cleaner.

Warranty

MTR+ Vertriebs GmbH provides a 24 month warranty, commencing from the date of our delivery (invoice date), that in case of failure, the **SineBravo** will be serviced or replaced free of charge. All returns must first be authorised by MTR+ Vertriebs GmbH in advance.

Excluded from this warranty are failures of the **SineBravo** which result from excessive use, careless or improper handling as well as damage by force.

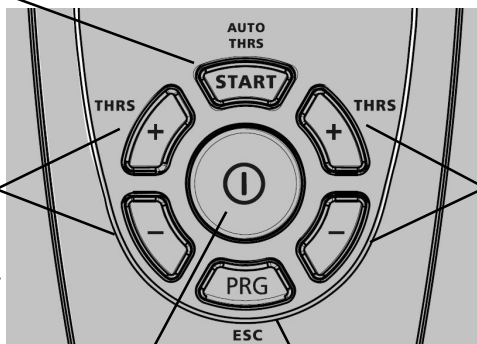
The warranty loses its lawful meaning if the producer MTR+ Vertriebs GmbH has not given its consent in writing prior to repairworks or servicing being carried out. Without this written consent the manufacturer will not return the device.

Operating Elements

The **START**-button allows the choice of automatically (short pressure) or manually (keep the button pressed for 3 seconds) threshold-setting and starts each program.

+/- buttons for threshold-setting on channel A as well as for selection of parameters in the programming-mode


+/- buttons for threshold-setting on channel B as well as for altering the parameters in the programming-mode

Central On-/Off-button

PRG-button for choice of program and for storing the individual parameters

Battery Compartment

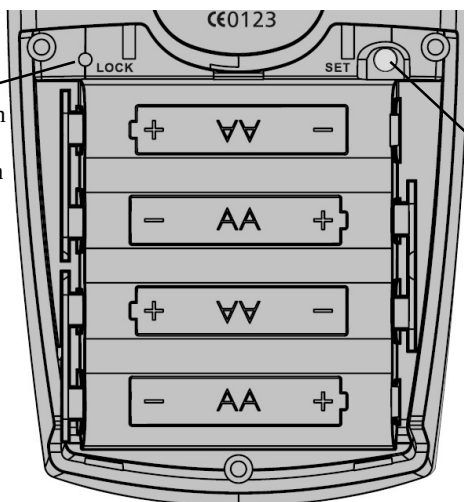LOCK-button for fixing one single program and for connection to a PC


SET-button for fundamental adjustments and configuration of each single phase

Scheme for placement of 4 batteries type AA

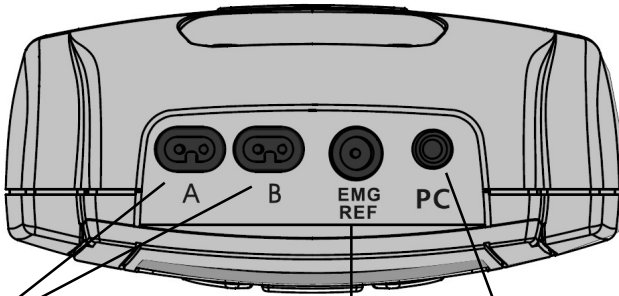Connections

Jacks to connect the electrode-cables of channel A and/or B, each suited for surface-electrodes or probes.

Jack for the fibre optic cable to connect via USB-adapter to PC

Jack for the single EMG-reference cable (skin-electrode)

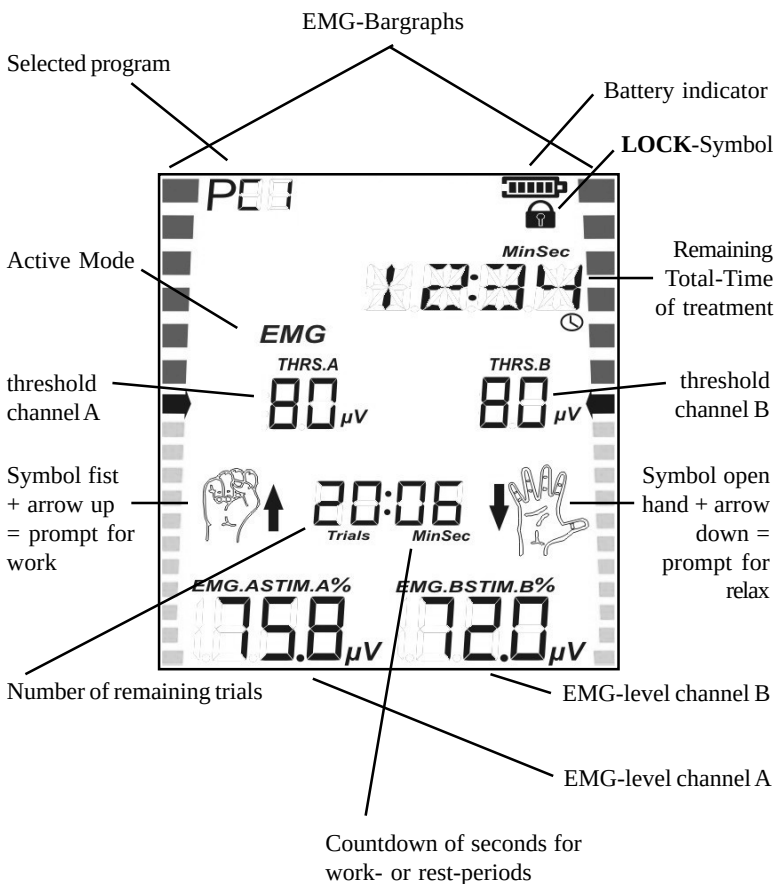
Attention: Even if the device is used with a probe, please always connect the single reference-cable.

The Display

After selecting a program with the **PRG**-button **SineBravo** shows in its "Home Screen" information about the parameters that are used in this program (number of trials, threshold, etc.). With the start of treatment by pressing **START**, the display changes to work-screen and the rhythm of work and rest becomes indicated.

Work-screen

The Info-Line

The info line in the upper part of the screen shows always the remaining time of the entire program during treatment.

Outside a treatment session, it provides much more additional information:

- If the circuit is interrupted, since an electrode has resolved or a cable is broken, the term CABLE flashes in the info line.
- While setting the clock or when choosing the language, it shows any value to be set in plain text.

Especially if the user tries to configurate the device, the info line is very helpful. In the following you find all expressions and abbreviations that are used in the programming-loop in alphabetical order:

CH.B. ON / CH.B. OFF: Channel B is switched on or off

FB. CONT: Continuous acoustic feedback in case of success

FB. OFF: No acoustic feedback in case of success

FB.ABOVE / FB.BELOW: Feedback above or below the threshold

INHIB.OF: Inhibition switched off

INHIB.ON: Inhibition of feedback on channel B

NARROW: Narrow bandpass filter used

PC ON / PC OFF: Connection to PC is switched on or off

REST: Duration of rest-sequences in seconds

THRESH: Threshold-adjustment during treatment automatically (AUTO) or manually (MAN)

TRIALS: Number of repetitions

VOLUME: Sound volume of beeps and spoken instructions

WIDE: Wide bandpass filter used

WORK: Duration of work-sequences in seconds

Basic Settings

In all programs, even in the preset programs, individual settings can be made as follows.

After you have put in four batteries type AA (s.p. 35) and you have turned on the device with the central **ON/OFF**-button, you enter the configuration mode by pressing the **SET** button in the battery compartment for three seconds.

Now you can adjust the settings for the first phase of the selected program. To configure the next phase, **SET** should be pressed again.

The parameter, that can be changed is shown in the info line and can be altered by pressing the + button of channel B. Pressing the + button of channel A will confirm the choice and the next variable parameter is displayed. This system - changing settings with B / storage and stepping forward with A - remains always the same during the entire process.

Please note that all the decisions you make here are automatically adapted for all subsequent phases and all other programs as well. Of course you can change any decision at any time in the same way again.

The following functions can be influenced:

- Sound-volume
- Automatical threshold-adjustment on or off
- Wide- or narrow-filter for channel A
- Channel B switched on or off
- Filter for channel B if it is in use
- PC-connection switched on or off

In the freely programmable programs PC1 - PC3 of course you have the chance to determine additional parameters in the same way as described before (s.p. 43-44).

Setup Real Time Clock

Before first use of **SineBravo** the “Real Time Clock” that is integrated should be set. It's absolutely necessary for later therapy-control. **SineBravo** has a small assistance battery that will support the clock with energy for several years. For this reason, there is no need of new setting, even if the 4 main batteries have been substituted.

For setting time and date, please insert 4 batteries type AA (s.p. 35), select any program and press the **SET**-button for at least 3 seconds. You'll reach the configuration-mode. Now press **SET** once again for 10 seconds. On screen HOUR is indicated and you can set the hour with the +button of channel B (00-23).

By pressing the +button of channel A you confirm this choice and step forward to the setting of MINUTE. This is indicated on the screen as well, giving the opportunity to select the minutes via +button of channel B in the same way as before.

This system - changing settings with B / storage and stepping forward with A - remains always the same. This way you are able to set one after the other:

- HOUR
- MINUTE
- DAY
- MONTH and
- YEAR

If the setting is finished, please press the **ESC**-button on the frontside of the device. All values will be automatically stored and the clock starts working.

Setup Language

SineBravo is able to show 6 different languages. If you are in the setting-mode of the RTC (see above), please press the **SET**-button for 10 new seconds once again. Now you can select German-, English-, French-, Spanish-, Italian- or Polish-language with the B-buttons. Finally you store your choice with the **ESC**-button.

Preparation of Therapy

1. Open the battery compartment at the back by pushing down the lid and insert 4 batteries type AA by hand according to the instructions on page 35. Then shut the batterie-compartment.
2. Please make sure, that **SineBravo** is switched off (no indication in LCD-display). Connect the single poled ends of the electrodes plugs with the surface-electrodes or with a probe.
3. Insert one electrode-cable into the jack named A in the head of the device. If you want to use both channels, insert the second cable into jack B.
4. Additionally, please put the black round EMG-reference plug into the bigger round jack, named “EMG-REF” and connect the other ends to a skin-electrode.
5. Furthermore, make sure that all plugs fit completely into the electrode sockets as well as the device sockets.
6. If you are using a probe, make it gliding capable with water or a gel suitable for electrotherapy. Don't use any oil, ointment or cream, since all these have an insulating effect.
7. Now place all electrodes according to the instructions of your therapist on clean unwounded skin only. If you are using a probe, place the single reference-electrode onto the hip, near the ankle first and then insert the probe.
8. Now switch on **SineBravo** by shortly pressing the central **ON/OFF**-button once. You can see the version of the device for 3 seconds before the last used program appears, showing its parameters.
9. If the device is not locked to one single program, you can select your kind of treatment by repeatedly pressing the **PRG**-button.
10. After selecting any program, you always see the “Home Screen” first, with all information regarding the parameters, that are contained in this program. There are two pre-set EMG-programs and three free configurable EMG-programs available.
11. The treatment starts with the automatic threshold-setting by pressing the **START**-button shortly. No further adjustments have to be done. If you prefer the manually threshold-setting, treatment starts by pressing and holding **START** for 3 seconds.

Therapy with SineBravo

SineBravo is a versatile device for biofeedback-treatment. Nevertheless it is the fundamental target, to be simple in use for the patient.

The device has two pre-set EMG-programs (P01 + P02), that can be used at once - without any further adjustment. But if you like to create your own treatment with individual parameters, you only have to select one of the three free places (PC1 - PC3). Here you have the chance to realize your own ideas.

Use of pre-set programs

1. Connect the electrodes to the device (s.p. 36)
2. Place electrodes or probe according to the instructions of your therapist
3. Switch on **SineBravo** with the central ON/OFF-button
4. Select the desired program with the **PRG**-button
5. Press **START** and normally threshold will be set automatically by doing two short work/rest-periods now. If that's not intended, you can switch off this function by holding the **START**-button for three seconds (s.p. 46) In this case you can set threshold manually with the +/-button of the respective channel.

The device indicates a pre-set rhythm of work and rest, that the patient shall execute as precise as possible.

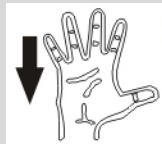
During work-period the symbol of the fist with arrow up is to be seen on the screen. The feedback regarding the success of your own muscular activities is given by the increasing numbers in the lowest line and by the yellow/green bargraph on each side of the display. Reaching the green area shows that the threshold - regardless whether it has been found out automatically or has been set manually - is met and the trial was successful. Additionally this success is indicated by signal beeps. Please take care to keep your efforts alive, until the time, that is indicated beside the fist, has gone to zero and the symbol of the open hand with arrow down appears. Then relax the muscle as far as possible.

If you have chosen automatical threshold adjustment during treatment, threshold would be reduced if you are not successful and it would be easier to succeed in the next trial.

If this next trial is successful again, threshold would newly increase.

The whole process of work and rest shall be repeated again and again - until the number of remaining trials is indicated 00 (s.p. 37). Please note that the display doesn't change before the relaxation-phase is completed.

If the relaxation is target of the treatment, the positive signal will be given in the active period (that is indicated by the fist) as well, if muscle tension will be below the threshold.

Additional Options for Configuration of SineBravo

Additionally to the handling of pre-set programs that is described in the previous chapter, **SineBravo** offers a great variety of possibilities to adapt the therapy-parameters according to individual needs.

This concerns the pre-set programs that can vary in regard of the general conditions as well as the 3 free customizable program-places, in which the experienced therapist or user is able to create completely new programs.

Configuration of Free Programs

Creating a new program is not difficult, because the programming always follows the same simple scheme. The device leads in a loop to each parameter that can be chosen - indicated by flashing bars.

1. Switch on **SineBravo** with the **ON/OFF**-button
2. Select any free place with the **PRG**-button. These places are indicated PC1-PC3 in the left upper corner of the screen.
3. Keep the **SET**-button in the battery-compartment pressed for 3-5 seconds. Then **SineBravo** changes to programming mode and the bars beside the first parameter that can be set are flashing.
4. In the following the right “+/-”buttons (channel B) are responsible for changing the flashing parameter, while the left “+” button (channel A) is used to store the choice and step forward to the next parameter. With the left “-”button it is possible to go one step back.

With the described scheme “right side changing / left side storing and step forward” all necessary decisions can be met. If the first parameter is shown a second time, the configuration of the whole program is complete and it can be stored with **ESC**. This configuration will stay available until a new programming is done on this place or the whole device is reset to its original state.

Parameters of Free Programs

PC1-3 are single-phase EMG-programs. With A+ the following parameters can be chosen and then altered by B+, if necessary:

- **Volume** - (indicated in display: VOLUME) - lays down the volume of sounds for feedback in steps 1-10
- **Duration of work-period (WORK)** - determines the time, that one single muscle contraction should be hold
- **Duration of rest-period (REST)** - determines the time of relaxing after contraction
- **Number of trials (TRIALS)** - names the number of active exercises, that shall be done
- **Threshold (THRESH)** - defines whether automatical threshold-adjustment during therapy is switched on (AUTO) or off (MAN).
- **Acoustic feedback** - you decide whether
 - a) you want to get only optic feedback (FB. OFF) or acoustic feedback as well (FB. ON) - and
 - b) the positive feedback shall be given if the patient will be above the threshold (FB.ABOVE - always in case of training the muscle-force) or below threshold (FB. BELOW - relaxation-training)
- **Wide or narrow filter (WIDE / NARROW)** for channel A - this choice is necessary to avoid wrong measurement, caused by heart frequency (60-120 Hz). If the patient measures EMG near the heart you should use NARROW-filter.
- **Channel B (CH.B. ON / CH.B. OFF)** switched on or off
- **Wide or narrow filter** for channel B
- **Feedback inhibition (INHIB.OF / INHIB.ON)** - will be selected, if the user shall get positive feedback only, if he will be above threshold of channel A and below threshold of channel B. This way it is tried to avoid wrong EMG-results, caused by wrong muscles, that are contracted near the measured muscles.
- and **PC-Interface (PC ON / PC OFF)** - here you can decide whether all treatment data will be transfered with the help of the optional therapist-software via fibre optic cable to a PC during treatment directly or not. During home-therapy PC OFF should always be selected to save battery-power.

Lock-Button

If the therapist wants to permit for his patient the use of one single program only, all other programs can be locked out.

To do this, please select the desired program and press the **LOCK**-button with a thin pin. For this target you can use the electrode connector. You'll hear a short beep and the symbol of a padlock appears on screen.

If you want to use any other program later, or if you like to unlock all programs, **LOCK** must be pressed in Home Screen once again and CONN-PC (= PC-connection) or NO.STATS (= no statistics) will appear. If you keep the **PRG** button pressed for 3 seconds now, the display changes to Home Screen and all programs are available. If you press **PRG** shortly, you return to Home Screen as well, but the program is still locked.

Threshold

The threshold is a value that shall be achieved by personal effort. It is measured in microvolts and displayed for both channels in the middle line of the screen - possibly even different. The arrows in the middle of the bargraph on the left and right side of the display show towards these thresholds. Since all programs refer to a threshold, it is necessary to start any treatment with the setting of this threshold.

Setting of threshold before start of treatment

By default, the threshold is determined automatically by switching on the device.

A short pressing of the **START** button causes the appearance of the open hand in the display, while the time-indicator for this rest-period runs to 00. At 00, the display switches to the symbol of the fist and the time for this 5-second period of work runs to zero again. **In this short work period, you should try to contract the muscle at the maximum possible effort.** Subsequently a rest period follows and a further work-period. During these two initial work/rest trials for threshold-detection the word TEST is indicated in the infoline. The threshold is calculated automatically after the second period of work, and is set at 80% of the average value of the period with higher performance. Now the treatment begins with the threshold determined this way and the total time counts down in the infoline.

Alternatively, the automatic setting can also be skipped. This is achieved by holding the **START** button pressed for 3-5 seconds. In this case, the threshold of the last treatment appears and could be altered manually by pressing the +/- buttons.

Automatical adjustment of threshold during treatment

Once the threshold is set, it will stay the same for the entire duration of the treatment until you make a new setting - except you have selected "automatic threshold adjustment." In this case, the threshold would be adjusted during the treatment according to your performance.

Each new threshold will be set to 80% of the average work that have been done in the last trial before.

Therapy-Control

SineBravo has a memory that can be read immediately after each complete treatment session. All values of the just finished session named in pages 48-49 will be indicated on the screen automatically. With A+ you can scroll through the statistics.

If it is intended to control therapy of several sessions, days, weeks or months, it is possible to receive the stored treatment data of the last 365 treatment days. If the additional software is available, of course the entire data can be transferred at any time without any problems to the PC. An electronic patient record, which allows the graphical presentation of the treatment progress, can easily be created. This optional software is very helpful for therapists for statistical and / or scientific reports. Please connect **SineBravo** using the software package accompanying USB cable to the PC and adjust PC ON at the device. If you press the **LOCK** button once, the display shows CONN-PC and the daily statistics will be transmitted.

If the device is locked in place (see page 45) and CONN-PC is displayed, you can read data about the average daily statistics on screen as well by pressing A+.

The values in the therapeutic memory are retained even after they are read, so it is possible to check therapy at any time, without losing information.

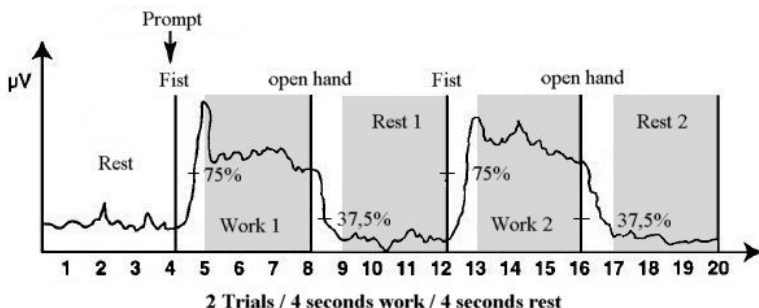
If the memory shall be deleted completely, it can be done with the following button-combination. Press the **LOCK** button in the battery compartment and then successively **SET**, B +, B- and **SET** once again. You will hear 6 beeps to confirm. This action will also reset all custom settings in the freely customizable programs back to the original state.

The deletion of the memory is required:

- before the device is used by a new patient
- 1 year after initiation of treatment at the latest, even at constant user.

Statistics

After termination of a treatment session **SineBravo** shows statistics on its screen as follows.

WORK.AVG - Work Average:

The average value in microvolts of all the work-segments excluding the first second of each segment.

REST.AVG - Rest Average:

The average value of the rest-segments excluding the first second.

ONSET - Average Onset:

The average time taken after each respective “**work**” prompt to reach 75% of the average value of the previous work period. If the onset of trial(s) is longer than 2 seconds, it will be rejected. The display will indicate the average of only those trials, where onset lasts 2 seconds or less. The number of valid onsets is displayed above the onset value. If the number of valid onsets is lower than the number of trials, it means some onsets were rejected.

RELEASE - Average Release:

The average time taken, after the “**rest**” prompt to reach 37.5% of the average EMG from the previous work period. If any release of the trials is longer than 2 seconds then it will be rejected. The display will indicate the average of only those trials where release lasts 2 seconds or less. The number of valid releases is displayed above the release value. If the number of valid releases is lower than number of trials, it means some releases were rejected. If all the releases are above 2 seconds, ABOVE2 will appear on the screen.

Statistics

W/R PEAK - Work/Rest Peak:

It is the peak value during the whole cycle of all the trials.

WORK DEV - Average deviation (per second) of work:

It is the average deviation of the quarter-second samples from the average value in the second in which they occur (excluding the first second of each work-segment). If deviation is more than 20% it is considered problematic.

REST DEV - Average deviation (per second) of rest.

The average deviation of the quarter-second samples from the average value in the second in which they occur (excluding the first second of each rest-segment). If deviation is more than 20% it is considered problematic.

Programs

Program	Nr.	No. of Trials	Work Time	Rest Time
EMG Urology	P01	40	8 Sek.	8 Sek.
EMG Neurology	P02	30	8 Sek.	12 Sek.
EMG free	PC1	2-99	2-99 secs.	2-99 secs.
EMG free	PC2	2-99	2-99 secs.	2-99 secs.
EMG free	PC3	2-99	2-99 secs.	2-99 secs.

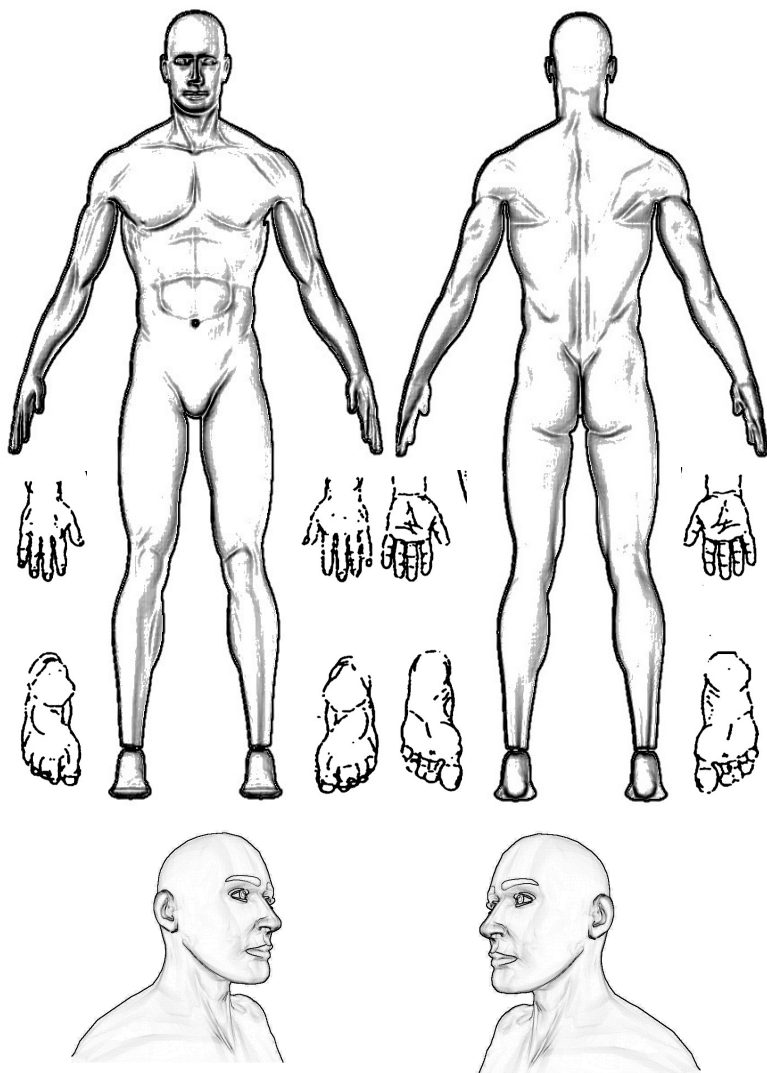
Literatur

1. **Armagan, O. / Tascioglu, F. / Oner, C.:** Electromyographic Biofeedback in the Treatment of the Hemiplegic Hand; *Am. J. Phys. Med. Rehabil* 2003; 82, 11, 856-861
2. **Aukee, P. / Immonen, P. / Laaksonen, D.E. / Laippala, P. / Penttinen, J. / Airaksinen, O.:** The effect of home biofeedback training in stress incontinence; *Acta Obstet. Gynecol. Scand.* 83, 2004: 973-977
3. **Berghmans, L.C. / Frederiks, C.M. / de Bie, R.A. / Weil, E.H. / Smeets, L.W. / van Waalwijk van Doorn, E.S. / Janknet, R.A.:** Efficacy of Biofeedback, When Included With Pelvis Floor Muscle Exercise Treatment, for Genuine Stress Incontinence; *Neurourol. Urodynam.* 15, 1996: 37-52
4. **Crow, J.L. / Lincoln, N.B. / Nouri, F.M. / de Weerd, W.:** The effectiveness of EMG- Biofeedback in the treatment of arm function after stroke; *Int. Disabil Studies*, 11, 1989, 155-160
5. **Davlin, C.D. / Holcomb, W.R. / Guadagnoli, M.A.:** The Effect of Hip Position and Electromyographic Biofeedback Training on the Vastus Medialis Oblique: Vastus Lateralis Ratio; *J. Athletic Training* 1999; 34(4): 342-349
6. **Draper, V.:** Electromyographic Biofeedback and Recovery of Quadriceps Femoris Muscle Function Following Anterior Cruciate Ligament Reconstruction; *Physical Therapy*, 70, 1, 1990
7. **Glavind, K. / Nøhr, S.B. / Walter, S.:** Biofeedback and Physiotherapy Versus Physiotherapy Alone in the Treatment of Genuine Stress Urinary Incontinence; *Int. Urogynecol. J.* 7, 1996: 339-343
8. **Huang, H. / Wolf, S.L. / He, J.:** Recent developments in biofeedback for neuromotor rehabilitation; *J. NeuroEngineering and Rehabilitation* 2006, 3:1
9. **Laycock, J. / Standley, A. / Crothers, E. / Naylor, D. / Frank, M. / Garside, S. / Kiely, E. / Knight, S. / Pearson, A.:** Clinical Guidelines for the Physiotherapy Management of Females aged 16-65 with Stress Urinary Incontinence, Chartered Society of Physiotherapy, London 2001
10. **Lucca, J.A. / Recchiuti, S.J.:** Effect of Electromyographic Biofeedback on an Isometric Strengthening Program; *Physical Therapy*, 63, 2, 1983
11. **Pages, I.H.:** Biofeedbacktraining bei weiblicher Belastungsharninkontinenz; *Phys Med Rehab Kuror* 2005; 15: 359-363
12. **Schulte-Frei, B.:** Sport- und Bewegungstherapie für den weiblichen Beckenboden. Alltagsrelevanz, Analyse und Therapie unter besonderer Berücksichtigung der neuromuskulären Ansteuerung; Dissertation, Köln 2006
13. **Sung, M.S. / Hong, J.Y. / Choi, Y.H. / Baik, S.H. / Yoon, S.:** FES-Biofeedback versus Intensive Pelvic Floor Muscle Exercise for the Prevention and Treatment of Genuine Stress Incontinence; *J. Korean Med. Sci.* 15, 2000: 303-308
14. **Wolf, S.L. / Binder-Macleod, S.A.:** Electromyographic Biofeedback Applications to the Hemiplegic Patient - Changes in Lower Extremity Neuromuscular and Functional Status; *Physical Therapy*, 63, 9, 1983

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Reserved for changes.

Printing or copy of this manual - also in an abridged version - is forbidden and can only be allowed with a written approval of “MTR+ Vertriebs GmbH”.

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